# CL100

**Multifunction telephone**  
**Téléphone multifonctions**  
**Multifunktionstelefon**

**English P2**

**Français P14**

**Deutsch P26**

!!! User manual to read carefully before start to use !!!

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**CAUTION:** this telephone is specifically designed for the people with hearing difficulties and is provided with a handset earpiece volume control for individual requirements. Due care must be taken by all users that the handset earpiece volume control is set to the lowest level acceptable by each respective user. Care should therefore be exercised to ensure that any naive users are familiar with this requirement.

Normal operating temperature for the product (e.g. 0°C ~ 40°C).

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
!!!Please refer to our website: [www.geemarc.com](http://www.geemarc.com) for an up to date user guide, as there may be important updates and changes you need to be aware of !!!


<b>Description</b> .....	<b>3</b>
<b>Setting up</b> .....	<b>5</b>
1- Connecting the telephone .....	5
2- Ringer volume setting .....	5
3- Tone/Pulse switch .....	6
4- Time break recall setting .....	6
5- Wall mounting .....	6
<b>Volume and tone adjustment</b> .....	<b>8</b>
1- Receiving volume and tone adjustment .....	8
2- Sending volume adjustment .....	9
<b>Using the phone</b> .....	<b>10</b>
1- Answer a call .....	10
2- Making a call .....	10
3- Last number redial .....	10
4- Recall, * & # buttons .....	10
5- Mute button .....	10
6- Storing telephone numbers .....	11
7- Dialling stored number .....	11
<b>General informations</b> .....	<b>12</b>

# Description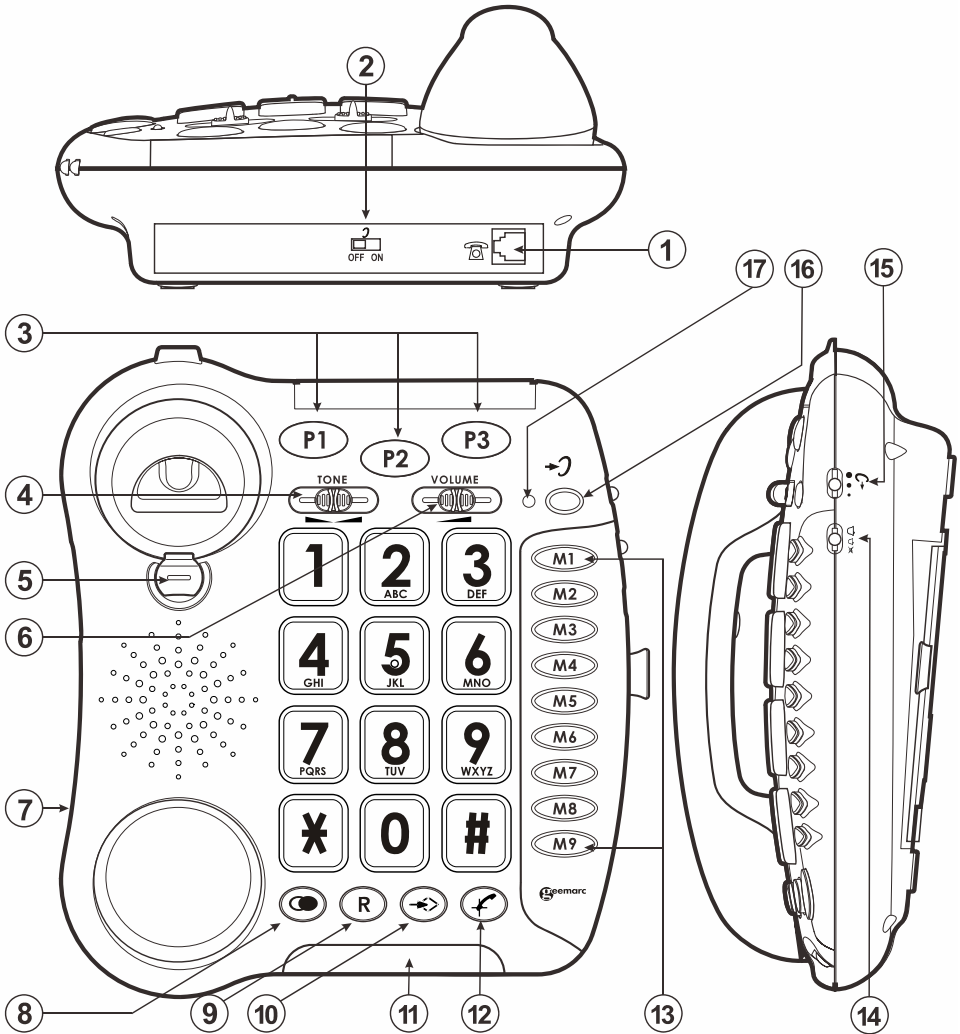

1. Line socket.
2. Volume and tone adjustment ON/OFF switch.
3. 3 emergency numbers.
4. Reception tone adjustment.
5. Clip for wall mounting.
6. Reception volumen adjustment.
7. Handset cord socket.

8. Last number redial.
9. Time break recall.
10. Memory storing.
11. Flashing incoming ring indicator.
12. Mute.
13. Direct memories.
14. Ring volume setting.
15. Sending volume setting.
16. Extra volume and tone adjustment.
17. Red light of tone/volume activation.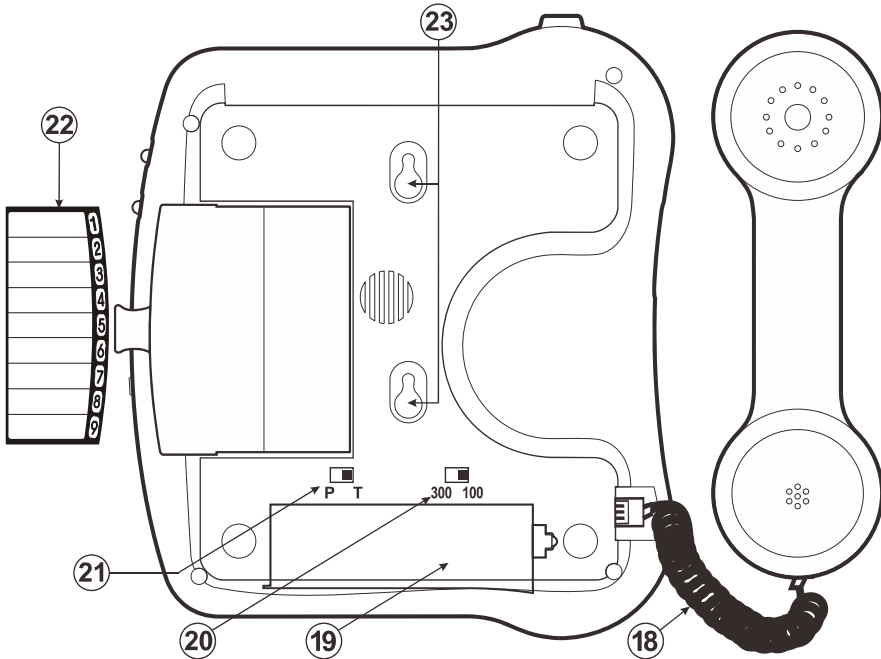


18. Handset cord.
19. Battery compartment.
20. Time break recall.
21. Dialling mode **P**ulse/**T**one.
22. Memo card.
23. Holes for wall mounting.

# Setting up

## 1 – Connecting the telephone

- Connect the handset cord (1).
- Open the battery compartment (2).
- Insert 4 alkaline batteries (type AAA 1.5V, not supplied) in the compartment (\*). Batteries are required if you wish to increase the ringer level up to 10 dB (2).
- Snap the battery door back in place.
- Connect the line cord plug into the line socket on the rear of the unit (\*) (3).
- Connect the other end of the line cord into the wall PTT socket (\*) (3).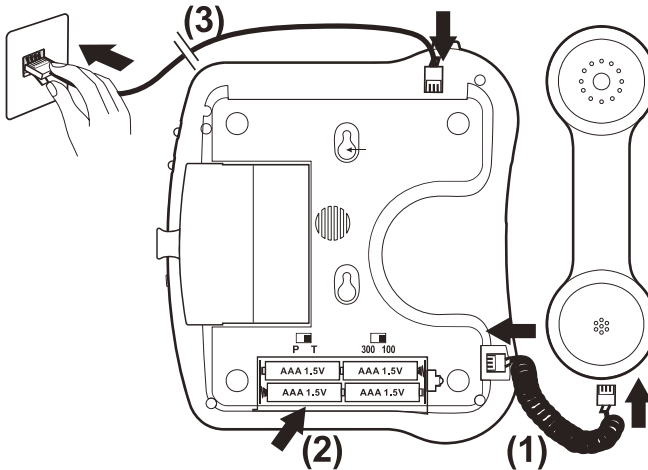

(\*) Classified TNV-3 according to EN62368-1 standard.

## 2 – Ringer volume setting

A switch located on the right side allows the ringer to be switched to Hi, Low or Off:

 : Off,

 : Low,


 : Hi.

**Note:** The ring level will be increased of 10 dB if you insert batteries.

## 3 – Dialling mode (Tone/Pulse) settings

---

The Tone/Pulse switch is factory pre-set to Tone position. In the UK, all telephone exchanges now use Tone (T) dialing. If your phone does not dial out, it is probably being used from an older private switch board (PBX). In this case, the Tone/Pulse switch must be changed to Pulse (P).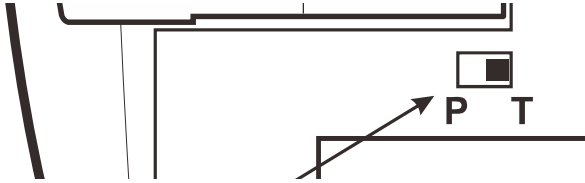

## 4 – Time Break Recall setting (300/100)

---


Your phone works with a time break recall for United Kingdom (R=100ms). It can be switched to 300ms for other locations or PBX's.

## 5 – Wall mounting

---


To wall-mount your telephone, turn the “clip” (located in the base unit, just below the ear piece).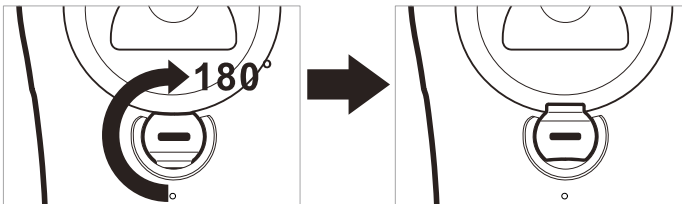

This will ensure that the handset stays firmly in position.

# Setting up

- Drill 2 holes in the wall (80mm apart).
- Insert the wall plugs and the screws leaving about 6 mm of the screws projecting from the wall.
- Slide the telephone downwards to secure.

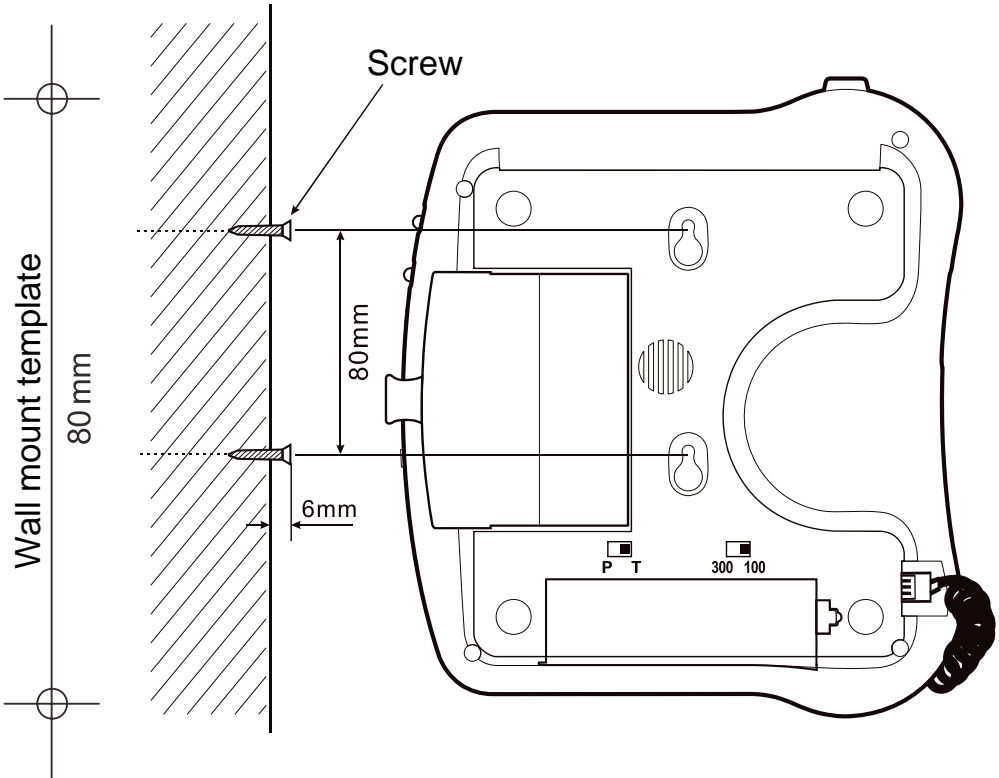# Volume and tone adjustment


## 1 – Receiving volume and tone adjustment

You can increase the volume and adjust the receiving tone according to your own hearing requirements.

The button  allows extra receiving and tone functions to be activated or not during the conversation.

When the function is activated, the red light of extra receiving and tone function located near the button  is lit.

Adjust the **TONE** and **VOLUME** buttons to fit the levels to your own hearing requirements.

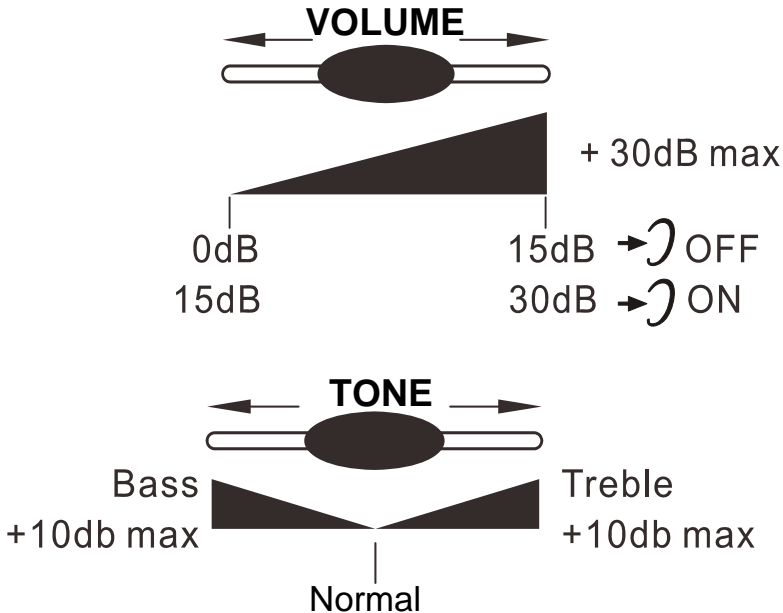**Note:** When the tone (treble), earpiece volume and sending levels are set to the maximum, whistling may occur. Then the sending level settings should be decreased with the switch

# Volume and tone adjustment

When the switch  located behind the phone is **ON**, the extra-receiving and tone functions are automatically activated each time you use the phone.

When this switch is **OFF**, you will need to activate the function by pressing the button .

Tone and volume (15dB to 30dB) adjustment is only activated when the button  is set to ON.


When this button is set to OFF only the volume can be adjusted from 0dB to 15dB.

## 2 – Sending volume adjustment

---

If your voice is quiet or called parties have difficulty hearing you, you can increase the sending level volume with the switch located on the right side.

You can also decrease the sending volume if required.

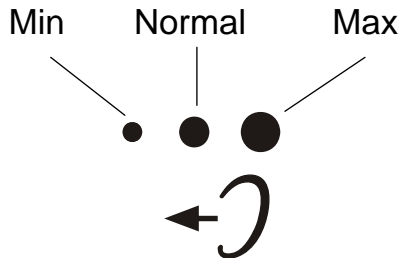# Using the phone

***If you are using this telephone with a hearing aid, set your hearing aid to the “T” setting.***

## **1 – Answer a call**

---

- When an incoming call is received the phone will ring and flash, to answer the call lift the handset and speak.
- On completion of the call, carefully replace the handset in the cradle to release the line.

## **2 – Making a call**

---

- Lift the handset.
- Await the dial tone and dial required number.
- On completion of the call, carefully replace the handset in the cradle.

## **3 – Last number redial**

---

- Lift the handset.
- Await the dial tone and press  button.
- The previously dialled number will be automatically redialled (not in use for memory numbers).

## **4 – Recall (R), \* & # buttons**

---

These are used with the new services provided on digital exchanges. For details please contact your network operator.

## **5 – Mute button**

---

If you wish to speak privately to someone else in the room without your caller overhearing, press the  button. You will still be able to hear the callers but they cannot hear you during the mute operation. Press  button again to resume normal conversation.

# Using the phone

## 6 – Storing telephone numbers

Your telephone can store 12 direct memories including 3 priority numbers (24 digits maximum for priority memory and 32 digits for **M1**, **M2**...**M9**). For example, the 3 priority **P1**, **P2**, **P3** memories allow you to store the 3 numbers below:

**P1** = Fire      **P2** = Police      **P3** = Doctor

To store a number please follow instructions below:

- Lift the handset.
- Press and hold  button, until hear “Di~” sound.
- Dial the number.
- Press  button again.
- Press **M1**, **M2**, ..., or **M9** or one of 3 priority memories to store your number.


**PLEASE NOTE:** If you hear a message advising “The number you are dialing has not been recognized, please check and try again”. Please ignore this and continue with the storing process and the numbers will be stored.

**Note:** Entering a new number in memory will automatically erase the previous one.

## 7 – Dialing stored numbers

- Lift the handset and await the dial tone.
- Press appropriate button **M1**, **M2**, ..., or **M9** or one of 3 priority memories, the stored number will be dialed automatically.

To register your stored phones numbers, use the index located under the base.

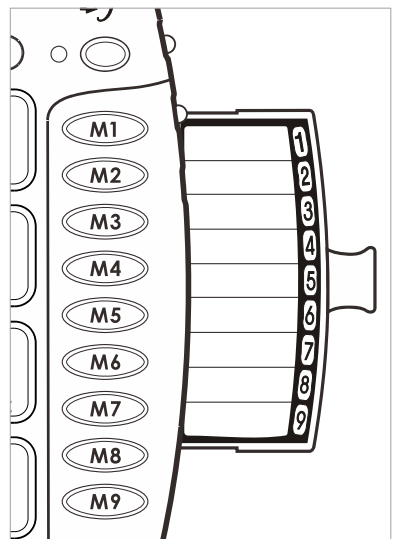## Recycling directives:

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products in end of life are recycled in the best way. When this product is out of order, please do not put it in your domestic waste bin.

Please use one of the following disposal options :

- Remove the batteries and deposit them in an appropriate WEEE skip.

Deposit the product in a appropriate WEEE skip.

- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

## Guarantee:

From the moment your Geemarc CL100 is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at **[www.geemarc.com](http://www.geemarc.com)**. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**Important:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**Telephone connection:** Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN62368-1 standard.

The telephone does not operate if the line current is lower than 18mA.

**Declaration:** Hereby Geemarc Telecom SA declares that this CL100 is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/30/EU and in particular article 3 section 1a, 1b and section 2.

# General informations

*The declaration of conformity may be consulted at  
[www.geemarc.com](http://www.geemarc.com)*

## Statement for Battery:

- **Caution:** Risk of explosion if battery is an incorrect type; Dispose of used batteries according to the instructions.
- The applied nameplate is located at the bottom the product.
- The mounting height of the equipment should be less than 2 meters.
- Avoid using battery in following conditions:
  - High or low extreme temperatures that a battery can be subjected to during use, storage or transportation;
  - Low air pressure at high altitude;
  - Replacement of a battery with an incorrect type that can defeat a safeguard;
  - Disposal of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery that can result in an explosion;
  - Leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas;
  - Extremely low air pressure that can result in an explosion or the leakage of flammable liquid or gas.

For product support and help visit our website at

**[www.geemarc.com](http://www.geemarc.com)**

E-mail : **[help@geemarc.com](mailto:help@geemarc.com)**

Telephone : **01707 387602**

lines are open 09h00 to 16h00 Mon to Fri

2 rue Galilée, Parc de l'Etoile, 59791 Grande-Synthe Cedex,  
France

This product is made in China To Geemarc Telecom

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